**OSSTEM**<sup>®</sup>  
IMPLANT

# Crestal Approach-Sinus KIT

## (CAS-KIT)

### User Manual

**OSSTEM**<sup>®</sup>  
IMPLANT

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### ■ Description

Osstem's Crestal Approach – Sinus KIT (CAS-KIT) is a tool KIT that is optimized for the fast and safe elevation of the maxillary sinus membrane in the Crestal Approach method for the maxillary sinus. Please refer to our product catalog or brochure for detail information.

### ■ Indication for use

A device intended for use in dental implant operation.

### ■ Directions for use

1. The exact height of the residual bone in the patient before the operation is known through CT.
2. Perform an initial drilling using Ø2.2 Twist Drill. At this time, it is recommended to fasten the stopper to perform '1mm under drilling' on the maxillary lower border. (1,200 RPM)
3. Then, according to the Drilling Sequence, the same stopper is fastened to the Final CAS Drill, and '1mm under drilling' is performed to enlarge the hole.
4. If the hole is enlarged to the final diameter, perform forward drilling by every 1mm in the direction of the maxillary sinus.
5. After drilling, be sure to use the Depth Gauge and Stopper to check the opening of the maxillary lower border.
6. If the lower border of the maxillary sinus has been fully drilled, you can proceed with the hydraulic pressure elevation, bone filling, and implant procedure.
7. For details, refer to the catalog or brochure.

### ■ Contraindications

Patients with the following conditions are contraindicated for surgical procedures :

1. Patients with coagulation disorders or bone/wound treatment disturbances.
2. Patients with uncontrolled diabetes, heavy smokers, or patients under the influence of alcohol.
3. Patients with decreased immune function due to chemotherapy or radiation therapy.
4. Patients with intraoral infection or inflammation.
5. Patients with untreatable occlusion/articulation problems and insufficient arch space.
6. Patients ineligible for surgery due to other reasons.

### ■ Precautions

1. Completely familiarize yourself with the manual before operation.
2. This product is a medical device, and should be disinfected before use as follows  
autoclave (Temperature : 132°C, Time : 15 minutes)

### ■ Caution in use

1. Only dentists are allowed to use this product. Do not use this product for other than the intended purpose.
2. Carefully inspect the product for any problem in external appearance or if it has been dropped on the floor. If there are any issues, then replace it with a new one.
3. In order to prevent the Stopper from being damaged, exercise caution to avoid loading excessive force during drilling.

4. Be sure to spray saline solution during to reduce the friction heat with the bone.
5. The recommended number of use for the drill and the driver is 50.

### ■ Caution after Use

1. Immediately separate all instruments used for the procedure, wash them thoroughly with flowing water using a cleaning brush, dry them, and store them at room temperature
2. Do not leave the instruments in a place vulnerable to contamination.
3. Do not use hydrogen peroxide as a disinfectant or cleaner, because it may cause damage or discoloration of TiN Coating, laser marking, color coding, and anodizing.
4. The warranty period for this product is 1 year after purchase.

### ■ Side Effect

A few problems may occur after the operation (loss of implant stability, damage of prosthesis, etc.). Deficient quality and quantity of the remaining bone, infection, allergic reaction, inferior oral hygiene or uncooperativeness of patient, implant mobility, partial deterioration of tissue, and improper position or arrangement of implants may cause the above mentioned problems.

### ■ General precautions

The surgical technique for a dental implant involves professional and complex procedures. Thus, the relevant professional education and training is required for implant operation.

### ■ Warnings

Inappropriate selection of patients and operation may cause implant failure or loss of supporting bone s. This product should not be used for purposes other than those prescribed by OSSTEM IMPLANT and should not be altered in any way. Implant mobility, bone loss, and chronic infection may result in the failure of an implant operation.

### ■ Product Information

The product name, specifications, manufacturing date, and product Lot are indicated on the package label.

※ For further details, refer to the CAS catalog and surgical manual provided by the salesman. For general information on implant, refer to the catalog and prosthesis manual.

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