# MS KIT

User Manual

**OSSTEM**<sup>®</sup>  
IMPLANT

**OSSTEM**<sup>®</sup>  
IMPLANT

Storage Condition :  
Dry place at room temperature(1°C~30°C)

Caution, Consult  
accompanying  
Documents

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TEL: 82-51-850-2500 FAX: 82-51-861-4693

<http://www.osstem.com>

**Rx only** For USA Only : Federal law restricts this  
device to sale by or on the order of a dentist

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## Manual for the MICRO SOLUTION (MS) IMPLANT KIT

### ■ About the Product

The MS Implant KIT contains Osstem MS Implant treatment tools for the restoration of defective teeth.

### ■ Cleaning the KIT

Clean the treatment tools after use as follows:

1. After treatment, clean the tools using the conventional method, i.e., by soaking them in alcohol.
2. After cleaning the tools with distilled water or under running water, remove any remaining blood stains and foreign matter completely.
3. Remove moisture completely using a dry cloth or a heat fan.
4. Insert and secure the moisture-free treatment tools into correct slots of the KIT.
5. After setting the tools, sterilize (132°C for 15 minutes) the KIT in an Autoclave and store at room temperature.

### ■ Cautions

1. When drilling, move the hand piece perpendicularly in an up and down (pumping) motion.
2. Be sure to spray saline solution during drilling to reduce the friction heat with the bone.
3. The Laser Marking indicates the drilling depth levels. The bottom line of the markings serves as the baseline. For the Laser Marking of all tools, 10mm and 11.5mm lines are marked Thickly.

[Figure. Baseline of Laser Marking]

4. The drill may be used for up to 50 times.
5. The driver may be used for up to 50 times. The maximum allowable torque of a 0.9 hex driver is 20Ncm, and that of a 1.2 hex driver, 35Ncm.
6. For the connection of a drill to the drill extension, let the flat handle tip face the opposite side of the slot. Avoid excessive force during the connection.
7. Separate all used tools immediately after the treatment and store them at room temperature after cleaning and drying.
8. Do not leave the instruments in a place vulnerable to contamination.
9. Be sure to sterilize these medical tools in the Autoclave before use. (132°C, 15 minutes)
10. Do not use hydrogen peroxide as disinfectant and cleanser (may cause damage or discoloration of the TiN coating, Laser Marking and color coding).
11. This product comes with a ONE (1) year warranty from the date of its purchase.

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