OneMS KIT

User Manual

OSSTEM[®]
IMPLANT

OSSTEM[®]
IMPLANT

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■ Description

OneMS KIT is a dedicated kit with Narrow specification for anterior teeth surgery using the OneGuide System. MS Ø2.0, Ø2.5, Ø3.0, and TSIII Ø3.0 fixture procedure is possible. In order to exclusively apply to anterior teeth cases, It is not possible to compatibly use with Tool & Guide hole of existing OneGuide System.

■ Indication for use

A device intended for use in dental implant operation.

■ Directions for use

1. Place the pre-made Guide in the patient's mouth.
2. Check the compatibility between tooth and Guide.
3. Once compatibility is confirmed, all the remaining surgical procedures can be performed once the Guide is fixed.
4. Remove the soft tissue from the site for implant fixation using the Tissue Punch (800 RPM).
5. Flatten the bone surface by using the Flattening Drill (1,200 RPM).
6. Using OneMS Drill, drilling is performed according to the specifications and bone quality of the fixture to be implanted (1,200 RPM).
7. Fix the fixture using the NoMount Driver, in which the direction of fixture hex needs to be aligned by matching the groove of driver with the Guide.
8. Adjust the final depth of fixation by connecting the Fixture

Driver to the Torque Wrench. The direction of the fixture hex needs to be aligned by matching the groove of driver with the Guide.

9. After surgery, remove the OneGuide Template from the mouth, and then follow the general implantation procedure for the rest of the steps.

■ Contraindications

Patients with the following conditions are contraindicated for surgical procedures :

1. Patients with coagulation disorders or bone/wound treatment disturbances.
2. Patients with uncontrolled diabetes, heavy smokers, or patients under the influence of alcohol.
3. Patients with decreased immune function due to chemotherapy or radiation therapy.
4. Patients with intraoral infection or inflammation.
5. Patients with untreatable occlusion/articulation problems and insufficient arch space.
6. Patients ineligible for surgery due to other reasons.

■ Precautions

1. Completely familiarize yourself with the manual before operation.
2. This product is a medical device. Users must follow the sterilization protocol below before use.
 - 1) Guide : Low-temperature sterilization
(Do not use autoclave or hydrogen peroxide.)

- 2) Other components : autoclave
(Temperature : 132°C, Time : 15 minutes)

■ Caution in use

1. Only dentists are allowed to use this product. Do not use this product for other than the intended purpose.
2. Carefully inspect the product for any problem in external appearance or if it has been dropped on the floor. If there are any issues, then replace it with a new one.
3. In order to prevent the Guide from being damaged, exercise caution to avoid loading excessive force during drilling.
4. Be sure to spray saline solution during to reduce the friction heat with the bone.
5. The recommended number of use for the drill and the driver is 50.

■ Caution after Use

1. Separate all used tools immediately after surgery. Store at room temperature after cleaning and drying.
2. Do not leave the instruments in a place vulnerable to contamination.
3. Guide is a disposable, non-sterile medical device and cannot be reused.
4. Do not use hydrogen peroxide as a disinfectant or cleaner, because it may cause damage or discoloration of TiN Coating, laser marking, color coding, and anodizing.
5. The warranty period for this product is 1 year after purchase.

■ Side Effect

A few problems may occur after the operation (loss of implant stability, damage of prosthesis, etc.). Deficient quality and quantity of the remaining bone, infection, allergic reaction, inferior oral hygiene or uncooperativeness of patient, implant mobility, partial deterioration of tissue, and improper position or arrangement of implants may cause the above mentioned problems.

■ General precautions

The surgical technique for a dental implant involves professional and complex procedures. Thus, the relevant professional education and training is required for implant operation.

■ Warnings

Inappropriate selection of patients and operation may cause implant failure or loss of supporting bone s. This product should not be used for purposes other than those prescribed by OSSTEM IMPLANT and should not be altered in any way. Implant mobility, bone loss, and chronic infection may result in the failure of an implant operation.

■ Product Information

The product name, specifications, manufacturing date, and product Lot are indicated on the package label.

※ For further details, refer to the OneMS catalog and surgical manual provided by the salesman. For general information on implant, refer to the catalog and prosthesis manual.

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DEUTSCHE OSSTEM GmbH