Sinus KIT

User Manual

OSSTEM[®]
IMPLANT

OSSTEM[®]
IMPLANT

Manufacturer

OSSTEM IMPLANT Co., Ltd.

66-16, Bansong-ro 513beon-gil, Haeundae-gu, Busan, Korea
TEL : 82-51-850-2500 FAX : 82-51-861-4693
<http://www.osstem.com>

Storage Condition :
Dry place at room temperature(1°C~30°C)

Caution. Consult
accompanying
Documents

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Rx only

For USA Only : Federal law restricts this
device to sale by or on the order of a dentist

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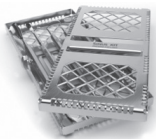
- Bone carrier with membrane elevator 1종(OBGC)

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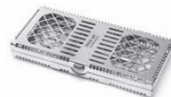
- Sinus Currettes(Short&Long) 각 1종(OSCS&OSCL)

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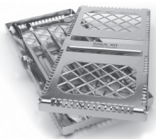
- Periosteal elevator :
Removal of maxillary bony window

- 1 Membrane separator & Bone packer (OMSC)

- Cylinder-type bone packer :
Filling bone grafting material in the maxillary sinus

- Membrane separator :
Initial exfoliation of the maxillary sinus mucosa

- 1 set (OBGC) of bone carrier with membrane elevator

- Spatula-shaped bone carrier :
For placing the bone grafting material and moving

- Narrow-tip membrane elevator :
Exfoliation and elevation of the maxillary sinus
mucosa

- 2 sets (OSCS&OSCL) of Sinus Curettes (Short & Long)

Use for the exfoliation and elevation of the maxillary
sinus mucosa.

Select a tip length depending on the oral environment.

- 1 KIT cassette set

Made of corrosion-resistant stainless steel; boasts of thermal safety during
sterilization.

■ Cleaning the KIT

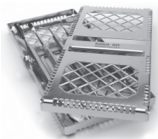
Clean the treatment tools after use as follows :

1. After treatment, clean the tools using the conventional method, i.e. by soaking them in alcohol.
2. After cleaning the tools with distilled water or under running water, remove any remaining blood stain and foreign matter completely.
3. Remove moisture completely using a dry cloth or a heater fan.
4. Insert and secure the moisture-free treatment tools into the correct slots of the KIT.
5. After setting the tools, sterilize(132°C for 15 minutes) the KIT in an Autoclave and store at room temperature.

■ Instructions on Usage and Cleaning

1. Detach all used instruments after operation and clean, dry, and keep them under normal temperature.
2. Do not leave the instruments in a place vulnerable to contamination.
3. Be sure to sterilize these medical tools in Autoclave before use.(132°C for 15 minutes)
4. Do not use hydrogen peroxide as disinfectant and cleanser(may cause damage or discoloration for the Laser Marking).
5. This product comes with a ONE(1) year warranty from the date of its purchase.

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DEUTSCHE OSSTEM GmbH